EN - Keep out of reach of children  
DE - Außerhalb der Reichweite von Kindern aufbewahren  
FR - Tenir hors de portée des enfants  
NL - Buiten bereik van kinderen houden  
IT - Tenere fuori dalla portata dei bambini  
ES - Manténgase fuera del alcance de los niños  
PT - Manter fora do alcance das crianças  
SE - Förraras utom räckhåll för barn  
DA - Opbevares utilgængelig for børn  
FI - Pidettävä poissa lasten ulottuvilta  
EL - Φυλάξτε μακριά τα παιδιά

U.S. Patents: 5,575,654; 5,863,202; 7,040,897; 7,056,118; 7,059,857; and 7,059,858; 7,568,579;  
7,625,210. Australia Patents: 654955 and 2004243104.  
Other U.S and International patents pending.

# UltraEZ<sup>®</sup> Flor-Opal<sup>®</sup>

EN- Tooth Desensitizing agents  
DE - Zahndesensibilisierungsmittel  
FR - Agents de désensibilisation des dents  
NL - Tanddesensitisiemiddelen  
IT - Agenti di desensibilizzazione dentaria  
ES - Agentes desensibilizadores para los dientes  
PT - Agentes de dessensibilização dentária  
SV - Medel för desensibilisering av tänder  
DA - Desensibiliserende midler til tænder  
FI - Hampaiden arkuutta vähentävät aineosat  
EL - Ουσίες απευαισθητοποίησης των δοντιών

800.552.5512 [ULTRADENT.COM](http://ULTRADENT.COM)

**ULTRADENT**  
PRODUCTS, INC.

EN DE FR NL IT ES PT SE DA FI EL

© Copyright 2010 Ultradent Products, Inc. All Rights Reserved.  
Manufactured by Ultradent Products, Inc. Made in USA  
505 West 10200 South, South Jordan, Utah 84095 41300.11 102010

# UltraEZ

## EN-Description:

UltraEZ<sup>®</sup> is a unique, sustained release, 3% potassium nitrate and 0.11% weight by weight fluoride ion, sticky, viscous gel. UltraEZ is designed to be used in a custom-fabricated tray.

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Lightly press into place.

Remove outer tray.

After placing tray in mouth, gently press the tray from biting surface of teeth towards the gums to ensure complete coverage of gel.

**UltraEZ** (Green Tray): Wear for 15 minutes to 1 hour per day or as directed by your dentist.

**Flor-Opal** (Blue Tray): Wear for 30 minutes to 1 1/2 hours per day or as directed by your dentist.

When finished remove tray and brush teeth.

### Precautions:

1. Keep desensitization product away from small children.
2. This product is NOT for nighttime use.
3. Do not use tobacco products or eat while using this product.
4. Not intended for use by pregnant women.
5. If any problems occur, discontinue treatment and notify your dentist.
6. Keep out of heat and/or sunlight at all times. UltraEZ and Flor-Opal may be stored at room temperature.
7. Patients should not swallow large volumes of UltraEZ or Flor-Opal gel.
8. Do not reuse to avoid cross contamination.

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