## Cleaning

General cleaning before sterilization: use a mild cleanser and soft brush or sponge, clean and rinse KlearView of any debris, blood or saliva before cold sterilization. Place elastic band around upper/lower cheek and lip retractor to reduce tension.

**U.S. federal law restricts this device to sale by or on the order of a dentist.**

**Keep out of reach of children.**

For immediate order and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call toll free: **1-800-552-5512**.

Outside U.S. call (801) 572-4200,  
or visit [www.ultradent.com](http://www.ultradent.com).

© Copyright 2003 Ultradent Products, Inc. All Rights Reserved.

Manufactured by: Ultradent USA  
505 West 10200 South, South Jordan, Utah 84095 USA

#31905.1 060503

**ULTRADENT**  
PRODUCTS, INC.

**KlearView™**

Cheek and Lip  
Retractor

***Very flexible and easy to use!***

### Indications:

To achieve working area accessibility. KlearView's transparent material is ideal for clinical photography.

### Instructions:

1. To facilitate patient comfort, apply lip balm or petroleum jelly to the patient's lips prior to inserting KlearView cheek retractors.
2. Hold both ends and squeeze the retractors together, gently bending the connecting bow in the cheek retractor.
3. The bow is positioned at the patient's chin and provides necessary tension to hold the device in place.
4. Instruct the patient to open his/her mouth. Insert the intraoral extension of each cheek retractor against the inside of the patient's cheeks. Gently guide the patient's lips to fit comfortably in the curve of each lip retractor.
5. To remove KlearView from the mouth, carefully grasp the left and right cheek handles. Perfect for lingual bonding with excellent view for upper arch.

Fig. 1

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EN Instruct the patient to open his/her mouth. Insert the intraoral extension of each cheek retractor against the inside of the patient's cheeks. Gently guide the patient's lips to fit comfortably in the curve of each lip retractor. To remove KleerView from the mouth, carefully grasp the left and right cheek handles. Perfect for lingual bonding with excellent view for upper arch.

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General cleaning before sterilization: use a mild cleanser and soft brush or sponge, clean and rinse KleerView of any debris, blood or saliva before cold sterilization. Place elastic band around upper/lower cheek and lip retractor to reduce tension.

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