U.S. Patent Numbers: 5,575,654; 5,863,202; 6,860,736;
6,964,571; 7,004,756; 7,040,897; 7,059,858; 7,128,899;
7,247,022; 7,481,653; and 7,568,579.

Australia Patent Numbers: 654955; 780274; 2004207495;
2004243104; and 2004251029.

Korea Patent Number: 10-0734581.

China Patent Number: ZL200480013948.5.

Russia Patent Number: 008520.

India Patent Number: 235284.

New Zealand Patent Number: 543633.

South Africa Patent Number: 2005/09141.

Other U.S. and International patents pending.

For an MSDS of this product, please see: www.ultradent.com

EN-Refrigerate
DE-Kühlen
FR-Réfrigérer
NL-Koelen
IT-Refrigerare
ES-Refrigerar
PT-Refrigerar
SV-Förvara i kylskåp
DA-Opbevares køligt
FI-Säilytä kylmässä
EL-Να φυλάσσεται
στο ψυγείο

800.552.5512 ULTRADENT.COM

 **ULTRADENT**
PRODUCTS, INC.

© Copyright 2010 Ultradent Products, Inc. All Rights Reserved. Made in USA

 Manufactured by Ultradent Products, Inc. 505 West 10200 South, South Jordan, Utah 84095 68536.9 102010

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'U' • Upper bleaching tray
'L' • Lower bleaching tray
Product must be refrigerated. Before using, allow product to warm to room temperature. This will make it easier for the tray to adhere to the teeth.
Remove from package.

Center tray on arch.

Gently suck down on the tray.

Remove outer tray.

Lightly tap tray if necessary.

Wear for 30-60 minutes per day. When finished remove bleaching tray and brush teeth.

Precautions:

1. Keep tooth bleaching product away from small children.
2. A dental examination is recommended prior to bleaching.
Some discolorations may indicate the need for dental treatment other than bleaching. Porcelain crowns, veneers, and white fillings do not lighten upon bleaching.
3. Only intended for use by those with healthy gum tissue. Gum tissue irritation may result if used in individuals with periodontal disease
4. This product is NOT for nighttime use.
5. Occasionally, some parts of teeth may bleach faster or less evenly than others. Results may be varied. In approximately 5% of the population, teeth may be resistant to bleaching. **BLEACHING TEETH WITH LARGE METAL FILLINGS MAY CAUSE THE METAL COLOR TO SHOW THROUGH THE TEETH.**
6. Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
7. Do not use tobacco products or eat while bleaching.
8. Not intended for use by pregnant women.
9. Keep gel out of heat/sunlight. Do not freeze.
10. Do not swallow gel or rinses. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.

WARNING: Some users may experience sensitivity, which if it occurs is most often mild and transient, but in some cases may require treatment by your dentist. If you experience significant discomfort, stop using this product and consult your dentist. If you have any questions about this product or treatment, please ask your dentist.

Do not reuse to avoid cross contamination.

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