U.S. federal law restricts this device to sale by or on the order of a dentist.

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For immediate reorder and/or complete descriptions of Ultradent's product line,
refer to Ultradent's catalog or call Toll Free **1-800-552-5512**.
Outside U.S. call (801) 572-4200
or visit www.ultradent.com

© Copyright 2003 Ultradent Products, Inc. All rights reserved.
U.S. Patent Number: 6,244,866 B1.

Manufactured by: Ultradent Products, Inc.
505 West 10200 South, South Jordan, Utah 84095

#24617.4 082503

ULTRADENT
PRODUCTS, INC.

PropGard™
Mouth Prop
with
Tongue
Guard

Description & Indications:

Use PropGard™ to hold mouth open and retract the tongue simultaneously.

Procedure:

1. Insert right or left tongue guard in mouth piece. Choose appropriate slot for superior/inferior relationship best suited to each patient.
2. Right and left noted on side of tongue guard. "R" or "L" notation should be to posterior in correct position in patients mouth.
3. Using index finger, press in on tongue guard to retract tongue as needed.

Fig. 1 Assemble PropGard as shown.

Fig. 2 Position PropGard in mouth.

Precautions:

1. Do not autoclave. PropGard is disposable.
2. Cold sterilization techniques can be used.

PropGard™

Mouth Prop/DE-Mundsperr /FR-Protège-langue/NL-Bijtblok/
IT-Apribocca/ES-Sostén Bucal/PT-Abre-Bocas/SV-Munöppnare/
DA-Mundspærre/FI-Purutuki/EL-Στοματοδιαστολέας

English

Insert right or left tongue positioner in mouth piece.

Precautions: 1. Do not autoclave. PropGard is disposable.
2. Cold disinfection can be used.

Using index finger; press in on tongue positioner to retract tongue as needed.

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