


Tissue Management Kit

1.800.268.1270
www.ultradent.com

Dento-Infusor[®] Tip

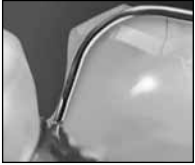


1. Rub ViscoStat[®] hemostatic firmly against bleeding tissues.



2. Firm air/water spray. Repeat step #1 if needed.

Ultrapak[®] Packer

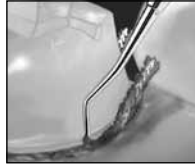


3. Pack Ultrapak[®] cord soaked in ViscoStat.



4. Firm air/water spray.



5. Remove cord. Firm air/water spray. Air dry. Make impression.

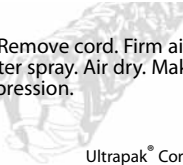

Ultrapak[®] Cord

© Copyright 2003 Ultradent Products, Inc. All Rights Reserved.
U.S. Patent Number: 4,551,100; 4,522,593; 4,895,517; 4,941,873;
and 5,635,162. European Patent Number: 0131360. Other U.S. and
International patents granted and pending.
#24242.3 120803