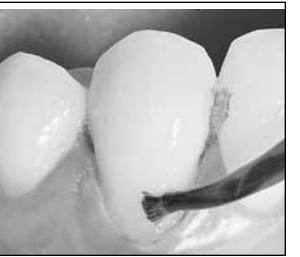
Composite Wetting Resin

D-Composite Wetting Resin/F-Agent de mouillage pour composite/ML-Compositebetøvsstigningshars/IT-Resina per bagnare I composite/F-Resina Humificadora para Composite/S-Vättningsresin för komposit/DK-Komposit Befugningsresin/FIN-Yhdistelmämuovin kostusresiliin/GR-Σβύβερτη πηκτική για δισκίη

Apply a thin layer to cured composite increment as needed. Air or vacuum thin (vacuum excess resin using 18-20 gauge Endo-Eze[®] tip on Luer Vacuum adapter).

If using Composite Wetting Resin for instrument glide, apply a small quantity to instrument or surface of soft composite.

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Composite Wetting Resin

Description and Indications:

Composite Wetting Resin is a 45% filled, radiopaque light cured wetting resin. It may be used during the placement of composite if the composite has become dry during incremental layering, or if the oxygen inhibition layer has been removed or disturbed. Composite Wetting Resin helps to improve the glide of the instrument during sculpting and contouring.

Procedure:

1. Remove luer lock caps and attach the Inspiral[®] Brush tip to syringe.
2. Apply a thin layer to cured composite increment as needed. Air or vacuum thin (vacuum excess resin using 18-20 gauge Endo-Eze[®] tip on Luer Vacuum adapter).
3. If using Composite Wetting Resin for instrument glide, apply a small quantity to instrument or surface of soft composite.

Precautions:

1. Any resin can be sensitizing to skin or can cause allergic reaction if individual is already sensitized. Therefore, wash area of skin exposed to resin completely to remove the material. If dermatitis, rash or any other allergic reaction is observed, see a physician.
2. When dispensing intraorally, never force syringe plungers. If there is excess resistance to flow, replace tip with a new one and verify flow before proceeding.
3. Use only recommended tips with each specific material. Tips are disposable.
4. We recommend using Ultradent[®] Syringe Covers sealed with the Ultradent[®] Impulse Sealer for a protective barrier to prevent cross-contamination.

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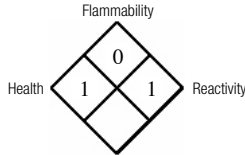
Composite Wetting Resin

HAZARD RATING
4 = Severe
3 = Serious
2 = Moderate
1 = Slight
0 = Minimal

MSDS Included.

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

Shelf life: 24 months. Store at room temperature.

U.S. federal law restricts this device to sale by or on the order of a dentist. For professional use only. Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free **1-800-552-5512**. Outside U.S. call (801) 572-4200.

© Copyright 2003 Ultradent Products, Inc. All Rights Reserved.
U.S. Patent Nos.: 4,997,371; 5,269,684; 5,534,562; and 5,708,052. International Patents Granted and Pending.

Manufactured by: Ultradent Products, Inc.
505 West 10200 South, South Jordan, Utah 84095
#23887.2 100903

ULTRADENT PRODUCTS, INC. MATERIAL SAFETY DATA SHEET			
SECTION I - IDENTIFICATION OF THE SUBSTANCE/PREPARATION AND OF THE COMPANY/UNDERTAKING		Controlled Product Yes <input type="checkbox"/> No <input checked="" type="checkbox"/>	
Material Name Composite Wetting Resin		Manufactured by Ultradent Products, Inc. 505 West 10200 South South Jordan, Utah 84095 USA	
Material Use Dental bonding resin		Emergency Contact 1-800-552-5512 or 1-801-572-4200 (Outside USA)	
Date November 20, 2000		EC Representative	
Prepared by: Steve Jensen, Chemist			
SECTION II - COMPOSITION/INFORMATION ON INGREDIENTS			
Active Ingredients	Approximate Concentration %	LD ₅₀ /LC ₅₀ Specify Species and Route	C.A.S. N.A. or U.N. Numbers
2-Hydroxyethyl methacrylate	15%	5.6 g/kg. Oral - Rat	868-77-9
SECTION III - HAZARDS IDENTIFICATION			
Adverse Human Effects and Symptoms Possible skin and eye irritation.			
SECTION IV - FIRST AID MEASURES			
Inhalation N/A	Eye Contact Flush with saline solution. Contact a physician.		
Skin Contact Wash with soap and water.	Ingestion N/A		
SECTION V - FIRE-FIGHTING MEASURES			
Flammability If Yes, Under Which Conditions: Avoid high temperatures and open flame.			
Means of Exinction CO ₂ , dry chemical.		Protective Equipment for Firefighters Not defined.	
SECTION VI - ACCIDENTAL RELEASE MEASURES		SECTION VII - HANDLING AND STORAGE	
Personal & Environmental Precautions Not defined.		Handling Precautions None.	
Methods for Cleaning Use dry cloth and alcohol.		Storage Requirements Room temperature.	
SECTION VIII - EXPOSURE CONTROLS/PERSONAL PROTECTION			
Respiratory Protection N/A		Eye Protection Protective eyewear.	
Hand Protection Latex gloves.		Skin Protection Lab coat.	
SECTION IX - PHYSICAL & CHEMICAL PROPERTIES			
Appearance Light yellow opaque resin.		Solubility in Water Insoluble. Melting Point/Range N/A	
Odor Methacrylate. pH N/A		Other Data None.	
SECTION X - STABILITY AND REACTIVITY			
Chemical Stability Conditions and Materials to Avoid Strong oxidizers.		Hazardous Decomposition Products CO ₂	
SECTION XI - TOXICOLOGICAL INFORMATION			
Route of Entry <input checked="" type="checkbox"/> Skin Contact <input checked="" type="checkbox"/> Skin Absorption <input type="checkbox"/> Eye Contact <input type="checkbox"/> Inhalation Acute <input type="checkbox"/> Inhalation Chronic <input type="checkbox"/> Ingestion			
Effects and Symptoms of Short-Term Exposure Eye: Irritation. Skin: Slight irritation.			
Chronic Effects and Symptoms of Long-Term Exposure Eye: Irritation and possible damage. Skin: Possible sensitization.			
Sensitization Slight.		Irritancy of Product Slight.	
SECTION XII - ECOLOGICAL INFORMATION		SECTION XIII - DISPOSAL CONSIDERATIONS	
Biodegradability Not defined.		Method of Waste Disposal In accordance with governmental regulations.	
Further Details None.			
SECTION XIV - TRANSPORT INFORMATION		SECTION XV - REGULATORY INFORMATION	
No limitations.		Not defined.	
SECTION XVI - OTHER INFORMATION			
FOR DENTAL USE ONLY. Use as directed. The information and recommendations are taken from sources (raw material MSDS(s) and manufacturer's knowledge) believed to be accurate; however, Ultradent Products, Inc. makes no warranty with respect to the accuracy of the information or the suitability of the recommendation and assumes no liability to any user thereof. Each user should review these recommendations in the specific context of the intended use and determine whether they are appropriate.			

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English	<div>General Precautions:</div> <ol style="list-style-type: none">For Professional use only. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed. Keep products out of heat/sunlight. Avoid skin exposure to resins. Isolate strong chemicals to area of treatment. Confirm that patient has no known allergies to treatment materials. Test flow of materials from syringe and tip before using intraorally. Never force syringe plungers. Use Ultradent[®] Syringe Covers and/or clean and disinfect syringes between patient. Use standard curing lights with all light polymerized/activated materials.
----------------------	--

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