# Flor-Opal®

Sustained Released  
Fluoride Gel

## Description:

Flor-Opal® is a unique, sustained-release, 0.5% fluoride ion in a sticky, viscous gel to be used with a custom tray. Flor-Opal is a clear, nearly tasteless, high-viscosity sticky gel with a pH of ~6.5.

## Indications:


Flor-Opal is designed for use whenever topical fluoride application is desired via take-home tray (e.g., prevention of root caries, treatment of periodontal conditions, following bleaching procedures, root sensitivity).

## Procedure:


1. Read and understand instructions before using or dispensing to patients.
2. Fabricate tray to meet patient's needs. See laboratory instructions (page 2) or send the working cast/model and prescription order form to Ultradent. Areas of sensitivity should be indicated on the cast before sending the model. If patient already has a custom bleaching tray, instruct them to apply Flor-Opal in the same manner as the bleaching gel.  
*Note: Trays made only for Flor-Opal may be constructed without scalloping; however, one should scallop trays made for Opalescence® bleaching.*
3. Flor-Opal is dispensed in 1.2ml unit dose syringes. An application of a complete arch should use most of the contents of one unit dose syringe. Tray is loaded by running gel on underside of occlusal and incisal areas.
4. The length and number of times the tray is worn depends on the condition, the patient, and the clinician. Recommended treatment times generally range from 2 to 8 hours. Wearing the tray at night may be suggested since the gel generally stays active in the tray 8-10 hours because of slower muscle and salivary activity at night. Gel usually lasts 4-6 hours with daytime activity.

**Ultradent Products, Inc.**  
**Attn: Tray Fabrication**  
**505 West 10200 South**  
**South Jordan, Utah 84095**

5. Tray should be removed at meal times. Following tray removal, patient should brush and rinse with water. DO NOT swallow solution during rinsing. Wash tray and store in a tray storage case. Fresh gel should be used for each application.
6. Gel may be warmed before placing in tray to avoid thermal sensitivity.

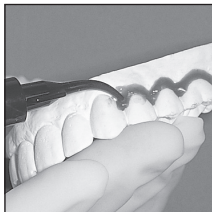*Fig. 1*

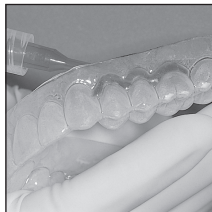*Fig. 2*

### **Laboratory Instructions:**

1. Pour and trim cast as you would for a standard bleaching tray. Apply 0.5-1.0mm of Ultradent® LC Block-Out Resin to a clean, dry cast.
2. Cure LC Block-Out for ~2 minutes in a light curing unit. A hand-held intraoral light (Ultra-Lume®) can be used (approximately 20-40 seconds per tooth). Wipe off oxygen inhibition layer.
3. With vacuum former (Ultra-Form® or EconoForm™), heat a Sof-Tray® sheet until it sags ~1 inch. Activate vacuum and adapt softened plastic over mold. Cool and remove model.
4. Cut excess bulk of material away with Ultra-Trim™ scalloping scissors.
5. Trim excess tray material. Contour the tray margin to avoid frenum attachments, tori, and similar anatomical structures.
6. Return tray to model; check tray extensions. Gently flame polish edges one quadrant at a time (Blazer® Micro Torch).
7. While still warm, hold periphery of each segment firmly against model for 3 seconds with water-moistened finger. If an area is short of the desired length, gently heat and push the tray material to the desired location. If this material thins too much, a new tray should be fabricated.

### **Precautions:**

1. If any problems occur, have the patient discontinue treatment and notify the dentist.
2. Patients should not swallow large volumes of Flor-Opal gel. Excess at insertion and residue upon removal should be brushed off and rinsed; DO NOT swallow.
3. Keep out of heat and sunlight at all times. Flor-Opal should be stored at room temperature.
4. Not intended for use by pregnant women.

**MATERIAL SAFETY DATA SHEET**

Controlled Product Yes  No

**SECTION I - IDENTIFICATION OF THE SUBSTANCE/PREPARATION AND OF THE COMPANY/UNDERTAKING**

Material Name <b>Flor-Opal<sup>®</sup></b>	Manufactured by Ultradent Products, Inc. 505 West 10200 South South Jordan, Utah 84095 USA
Material Use For treating tooth surfaces with fluoride.	Emergency Contact 1-800-552-5512 or 1-801-572-4200 (Outside USA)
Date July 17, 2006	EC Representative
Prepared by Richard Tuttle, D.D.S.	

**SECTION II - COMPOSITION/INFORMATION ON INGREDIENTS**

Active Ingredients	Approximate Concentration %	LD <sub>50</sub> /LC <sub>50</sub> Specify Species and Route	C.A.S. N.A. or U.N. Numbers
Sodium Fluoride	1.1% w/w	LD <sub>50</sub> 52mg/kg Rat-Oral	7681-49-4

**SECTION III - HAZARDS IDENTIFICATION**

Adverse Human Effects and Symptoms Sodium fluoride in large dosages is toxic.

**SECTION IV - FIRST AID MEASURES**

Inhalation N/A	Eye Contact Flush with lots of water.
Skin Contact Wash with soap and water.	Ingestion Give large amounts of water or milk.

**SECTION V - FIRE-FIGHTING MEASURES**

Flammability  If Yes, Under Which Conditions:  
Yes  No

Means of Extinction N/A	Protective Equipment for Firefighters N/A
----------------------------	--

**SECTION VI - ACCIDENTAL RELEASE MEASURES**

Personal & Environmental Precautions  
N/A  
Methods for Cleaning  
Wipe up with cloth and water.

**SECTION VII - HANDLING AND STORAGE**

Handling Precautions  
N/A  
Storage Requirements  
Room Temperature.

**SECTION VIII - EXPOSURE CONTROLS/PERSONAL PROTECTION**

Respiratory Protection N/A	Eye Protection Protective eyewear.
Hand Protection Latex gloves.	Skin Protection Lab coat.

**SECTION IX - PHYSICAL & CHEMICAL PROPERTIES**

Appearance Clear bubbled gel.	Solubility in Water Completely.	Melting Point/Range Not defined.
Odor Neutral.	pH ~6.5	Other Data None.

**SECTION X - STABILITY AND REACTIVITY**

Chemical Stability Conditions and Materials to Avoid  
Yes  No  Oxidants and strong acids.

Hazardous Decomposition Products  
N/A

**SECTION XI - TOXICOLOGICAL INFORMATION**

Routes of Entry  Skin Contact  Skin Absorption  Eye Contact  Inhalation Acute  Inhalation Chronic  Ingestion

Effects and Symptoms of Short-Term Exposure  
Eyes: Irritation. Ingestion: Irritation.

Chronic Effects and Symptoms of Long-Term Exposure  
Eyes: Irritation. Ingestion: Irritation (if ingested in large quantities).

Sensitization  
Not defined.

Irritation of Product  
Slight.

**SECTION XII - ECOLOGICAL INFORMATION**

Biodegradability  
Yes.

**SECTION XIII - DISPOSAL CONSIDERATIONS**

Method of Waste Disposal  
In accordance with governmental regulations.

Further Details  
None.

**SECTION XIV - TRANSPORT INFORMATION**

Not defined.

**SECTION XV - REGULATORY INFORMATION**

Not defined.


**SECTION XVI - OTHER INFORMATION**

FOR DENTAL USE ONLY. Use as directed.


The information and recommendations are taken from sources (raw material MSDS(s) and manufacturer's knowledge) believed to be accurate; however, Ultradent Products, Inc. makes no warranty with respect to the accuracy of the information or the suitability of the recommendation and assumes no liability to any user thereof. Each user should review these recommendations in the specific context of the intended use and determine whether they are appropriate.

Construct and dry the stone model, cutting palate and tongue areas out.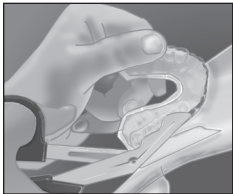

Vacuum adapt Sof-Tray® .035" sheet to model. Cool and separate.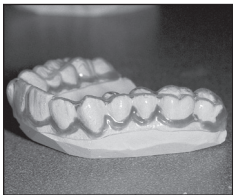

For sensitive roots use Ultradent's LC Block-Out Resin on gingival margin to provide a reservoir space for Flor-Opal. It is recommended to lap tray 2-3mm onto gingiva.

### General Precautions:


1. For professional use only.
2. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
3. Keep products out of heat/sunlight.
4. Avoid skin exposure to resins.
5. Isolate strong chemicals to area of treatment.
6. Confirm that patient has no known allergies to treatment materials.
7. Test flow of materials from syringe and tip before using intraorally.
8. Never force syringe plungers.
9. Clean and disinfect syringes between patients.


Load gel into your bleaching tray made by your dentist.  
Use 1/3 to 1/2 of syringe.

Brush teeth, then insert tray. Adapt tray sides to teeth.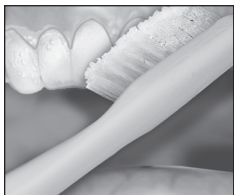

Remove excess gel with clean finger or soft toothbrush.  
Rinse twice; do not swallow rinses.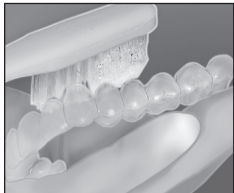

Clean tray with soft brush and cool tap water.  
Store tray in case provided.

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Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

## Flor-Opal

Flammability		HAZARD RATING	
	0		4 = Severe
Health	1	1	3 = Serious
		Reactivity	2 = Moderate
			1 = Slight
			0 = Minimal

### **MSDS Included.**

---

For professional use only.  
U.S. federal law restricts the sale of this device by or on the order of a dentist.  
Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line,  
refer to Ultradent's catalog or call Toll Free **1-800-552-5512**.  
Outside U.S. call (801) 572-4200  
or visit [www.ultradent.com](http://www.ultradent.com).

© Copyright 2006 Ultradent Products, Inc. All Rights Reserved.  
U.S. Patent Nos.: 5,234,342; 5,667,386; 5,759,037; 5,846,085; 5,851,512; 5,855,870; 5,985,249; 6,036,943; 6,086,855.  
Other U.S. and International Patents Pending.

---

Ultradent Products, Inc.  
505 West 10200 South,  
South Jordan, Utah 84095, USA

 **ULTRADENT**  
PRODUCTS, INC.

#10115.3 070706