**MSDS Not Required.**

HAZARD RATING 4 = Severe 3 = Serious 2 = Moderate 1 = Slight 0 = Minimal

**Key:**

Use By Date

See Instructions

Batch Code

Catalog Number

Do Not Reuse to Avoid Cross Contamination

**ULTRADENT**  
PRODUCTS, INC.

InterGuard®


**Description & Indications:**

InterGuard® is a preventive aid for operative dentistry. It is used to protect adjacent teeth from iatrogenic preparation damage. InterGuard is designed for autoclave and dry heat sterilization. Made of stainless steel with a curl at each end, InterGuard fits snugly in both large and small interproximal spaces. InterGuard is available in two heights:

- 4.0mm for short anatomical crowns.
- 5.5mm for tall anatomical crowns.

**Procedure:**

1. To prevent patient from accidentally swallowing InterGuard use a rubber dam or fasten a piece of dental floss through the hole on the side of InterGuard. To aid in retrieval make sure that the floss hangs no less than 7-inches outside of mouth.
2. Adjust the space between the curls with your fingers to allow InterGuard to fit firmly between the teeth.
3. Place InterGuard with curls toward the tooth to be prepared (Fig. 1). The curls keep the InterGuard clear of the transition angles, allowing full access to the area to be prepared.
4. With tight proximal contacts, use wedges to simplify placement or separate with a retaining ring.  
Note: At the end of preparation, InterGuard will loosen. If necessary, stabilize with a finger or InterGuard may be stabilized during the entire procedure with a retaining ring (in this case turn curls away from the tooth being prepared).

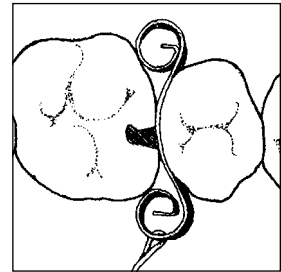**Fig. 1** Place InterGuard curls toward the tooth to be prepared.

**Precautions:**

1. Always isolate the preparation area with a rubber dam or ligate InterGuard with a 7-inch length of dental floss to prevent aspiration.
2. For professional use only.

U.S. Federal law restricts the sale of this device by or on the order of a state licensed dental professional.  
For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free **1-800-552-5512**.  
Outside U.S. call (801) 572-4200.


International Patents: Switzerland 122242, Denmark MR 0797 1995, Norway 72676, Sweden 59190, Germany M 9603438.2

800.552.5512 ULTRADENT.COM

**ULTRADENT**  
PRODUCTS, INC.

© Copyright 2010 Ultradent Products, Inc./InterGuard ApS. All Rights Reserved.  
Manufactured for Ultradent Products Inc. Made in Denmark 65432.8 101210

# InterGuard®

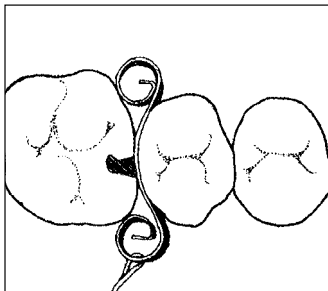**English**

To prevent aspiration, use rubber dam or dental floss (18cm).

Place the InterGuard between teeth. Adjust the curls for a firm fit. With tight proximal contact, separate with a wedge or a retaining ring

Place InterGuard with curls turned to tooth being prepared.

At the end of the preparation InterGuard will loosen. If necessary stabilize with a finger or stabilize during the whole procedure with a retaining ring (in this case turn the curls away from the preparation). Do not reuse to avoid cross contamination.

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