

Get More Confidence in Your Smile



You know you've been thinking about it. You know you want to.

Every time you pass the toothpaste aisle in the grocery store, you look for the simple cure for your dimming teeth. You want a whiter smile.

Good news. Now you can have one with very little effort. Introducing treswhite supreme, the easiest, fastest and best tasting way to put the white back in your smile.


Opalescence®
trèswhite™
••••• **SUPREME**

Manufactured by:


WARNING: Some users may experience sensitivity. If this occurs, it is most often mild and transient, but in some cases may require treatment by your dentist.

If you experience significant discomfort, stop using this product and consult your dentist. If you have any questions about this product or treatment, please ask your dentist.

© 2009 Ultradent Products, Inc. All rights reserved.
U.S. Patent Numbers: 5,575,654; 5,863,202; 6,860,736; 6,964,571; 7,004,756; 7,040,897; 7,059,858; 7,128,899; 7,247,022; 7,481,653; and 7,568,579.
Australia Patent Number: 654955; 780274; 2004207495; 2004243104; and 2004251029.
Korea Patent Number: 10-0734581.
China Patent Number: ZL200480013948.5.
Russia Patent Number: 008520.
India Patent Number: 235284.
New Zealand Patent Number: 543633.
South Africa Patent Number: 2005/09141.
Other U.S. and international patents pending.
For Professional use only.
For an MSDS of this product, please see: www.ultradent.com
68537.5 050510
Made in U.S.A.

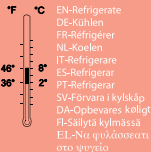

INSTRUCTIONS

PROFESSIONAL WHITENING MADE EASY

Opalescence®
trèswhite™
••••• **SUPREME**

INSTRUCTIONS



'U' • Upper bleaching tray
'L' • Lower bleaching tray
Remove from package



1. Center tray on arch.



2. Gently suck down on the tray.



3. Remove outer tray.



4. Lightly tap tray if necessary.



5. Wear for 30- 60 minutes per day. When finished remove bleaching tray and brush teeth.

PRECAUTIONS

1. Keep tooth bleaching product away from small children.
2. A dental examination is recommended prior to bleaching. Some discolorations may indicate the need for dental treatment other than bleaching. Porcelain crowns, veneers, and white fillings do not lighten upon bleaching.
3. Only intended for use by those with healthy gum tissue. Gum tissue irritation may result if used in individuals with periodontal disease.
4. This product is NOT for nighttime use.
5. Occasionally, some parts of teeth may bleach faster or less evenly than others. Results may be varied. In approximately 5% of the population, teeth may be resistant to bleaching. **BLEACHING TEETH WITH LARGE METAL FILLINGS MAY CAUSE THE METAL COLOR TO SHOW THROUGH THE TEETH.**
6. Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
7. Do not use tobacco products or eat while bleaching.
8. Not intended for use by pregnant women.
9. Keep gel out of heat/sunlight. Do not freeze.
10. Do not swallow gel or rinses. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.