Ultra-Etch[®]

35% Phosphoric Acid

Description:


UltraEtch[®] is a 35% phosphoric etchant solution with an optimum viscosity. The flowability allows precise placement, including occlusal grooves, yet is just viscous enough to prevent migration. UltraEtch[®] is self-limiting in its depth of etch (average depth of 1.9 μ with 20 second etch). This etchant contains no glycerin and is designed for rapid and complete removal upon rinsing.

Indications for Use:

For use with bonding adhesives to etch prior to placing composites and sealants.

Directions for Use:

1. For 1.2 syringe
 - a. Remove Luer cap.
 - b. Securely attach working tip of choice (Blue Micro[®] or Inspirial Brush[®] tip).
 - c. Verify etchant flow prior to applying intraorally.
2. For IndiSpense Syringe (Figures 1 & 2)
 - a. Remove Luer cap from Indispense syringe.
 - b. Attach 1.2 syringe to the male threads of the IndiSpense syringe.
 - c. Depress IndiSpense plunger while guiding 1.2ml syringe plunger to desired fill.
 - d. Separate syringes and re-cap IndiSpense syringe with Luer cap.
 - e. Securely attach working tip (Blue Micro[®] or Inspirial Brush[®] tip) to 1.2ml syringe.
 - f. Verify etchant flow prior to applying intraorally.
3. Direct placement instructions
 - a. Isolate tooth (teeth).
 - b. Use pumice, disk or diamond bur on uncut enamel.
 - c. Rinse and dry prepped area.
 - d. Verify etchant flow prior to applying intraorally.
 - e. Apply etchant to enamel and dentin for 20 seconds.
 - f. Rinse thoroughly, dry and proceed per adhesive manufacturer's instructions.
4. Restorative phosphoric acid cleansing
 - a. Apply etch to inside of porcelain veneer or crown for 5-10 seconds.
 - b. Rinse and dry.

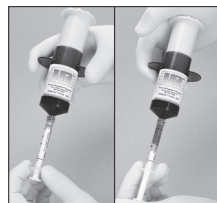Figs. 1 & 2 A 1.2ml unit dose syringe is twisted firmly onto the IndiSpense syringe. While supporting plunger of unit dose syringe with non dominant hand, depress plunger of IndiSpense and fill unit dose syringe to desired level.

Precautions and Warnings:

1. Carefully read and understand all instructions before using.
2. Keep out of reach of children.
3. If product comes in contact with any soft tissue, IMMEDIATELY rinse area with copious amounts of water.
4. Near pulp exposures should be treated with a protective base prior to placing etchant.
5. Re-cap, disinfect and wipe syringe with an intermediate level disinfectant between uses.
6. Dispose of tip and empty syringe properly.
7. Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips.

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
HAZARD RATING 4 = Severe 3 = Serious 2 = Moderate 1 = Slight 0 = Minimal

**For MSDS and additional information
about using Ultra-Etch, please go to
www.ultradent.com**

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts the sale of this device by or on the order of a dentist.
Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line,
refer to Ultradent's catalog or call Toll Free **1-800-552-5512**.

Outside U.S. call (801) 572-4200
or visit www.ultradent.com.

© Copyright 2003 Ultradent Products, Inc. All Rights Reserved.

Manufactured by:
Ultradent Products Inc.
505 West 10200 South
South Jordan, Utah 84095 USA

Made in USA

UL **ULTRADENT**
PRODUCTS, INC.

18005.6 010411