# Opalescence®

10%, 15%, 20%

## Tooth Whitening Systems AntiCaries

Opal PF 10% NPN : 80009556

Opal PF 15% NPN : 80008433

Opal PF 20% NPN : 80016256

Toothpaste NPN : 80002447

### Description:

Opalescence® is a dentist-controlled, dentist-supervised, take-home bleaching system dispensed in unit dose syringes. It is a clear, flavored, high-viscosity, sticky, 10, 15, or 20% carbamide peroxide gel (pH ~6.5). Opalescence has also been proven to reduce caries susceptibility, lower sensitivity, improve the microhardness of the enamel and improve enamel health.

Opalescence® PF™ carbamide peroxide gel contains potassium nitrate and 0.25% weight to weight fluoride. Opalescence PF is available in 10, 15, or 20% carbamide peroxide.

All Opalescence products have unique qualities including sustained release, adhesive properties, and a custom designed application tray. All Opalescence products are gluten free.

### Indications:

Opalescence has been shown to be effective for removing the colors present on teeth from the time of tooth eruption and/or the stains of aging. Success of varying degrees is usually obtained with tetracycline and brown fluorosis discoloration. Opalescence is also used on non-vital teeth for intracoronary bleaching. Opalescence provides a more conservative modality of treating dark, discolored teeth (compared to crowns, veneers, etc.). This includes discolorations caused by congenital, systemic, metabolic, pharmacological, traumatic, or iatrogenic factors such as dental fluorosis, tetracycline and adult minocycline stains, trauma, erythroblastosis fetalis, jaundice, and porphyria. Peroxide is a strong oxidizing agent. Bleaching occurs from the peroxide penetrating into the enamel and dentin and oxidizing the offending stains within the tooth.

Whitening occurs first and more rapidly within the enamel. For example, most of the disfiguring stains for tetracycline occur within the dentin; hence, their oxidation (whitening) requires more time. Because restorative materials will not whiten, we recommend whitening dark teeth before restorative placement so that restorations can be matched to the new lighter shades (wait two weeks following bleaching procedures before placing restorations). The dentist can now restore natural teeth to the lighter colors they used to be, or possibly to lighter natural shades never before experienced by the patient.

### General Information:

More attention must be given to tray design because of the sticky, viscous, sustained release features in comparison to early bleaching products! To avoid or reduce irritation, it is advised to limit peroxide contact to tooth surfaces rather than spilling onto soft tissues. If instructions are followed, more predictable results are obtained in days rather than weeks. Less peroxide is swallowed as loading of tray occurs one to two times daily. Gel stays active 8-10 hours in the mouth. Results occur more rapidly, which reduces the chronic long-term exposure to peroxide. Our experience has shown that a soft, thin tray material is best (Soft-Tray® 0.035") since it is more comfortable. With a thin tray, awareness of the tray is minimized since it occupies less of the patient's interocclusal space. In addition, scalloping and reservoirs further reduce pressure on the teeth and gums.

### Pre-Treatment Procedure:

1. Before bleaching treatment, a diagnosis must be performed. Ensure the presence of soft tissue health to minimize tissue trauma. Failed restorations and/or areas of caries should be restored. Exposed root surfaces may experience sensitivity. If large areas are exposed, or if restorations are inadequate, patients can develop mild to moderately severe pain. Sometimes this can be addressed by simply trimming the tray back, or areas can be covered with a bonding agent.
2. Remove calculus and external stains. If tissue is traumatized, wait one to two weeks before beginning bleaching treatment to minimize possible gingival sensitivity.

3. Wait two weeks following the bleaching procedure before matching and placing resin-bonded restorations because time is required for color stabilization. This is important before placing definitive tooth-colored restorations.
4. Occasionally a slight purple discoloration may appear in the tray following bleaching adjacent to amalgam fillings. This discoloration has proven inconsequential in our years of bleaching experience.
5. When bleaching teeth with large amalgam restorations or cores, the teeth may appear darker. This is because the internal restoration becomes more visible through the bleached enamel. The patient should be made aware that a more extensive esthetic restoration may be required following bleaching.
6. Some teeth do not respond to bleaching, and occasionally the colors return shortly after completion of treatment. This is often the result of a temporary color shift caused by dehydration (and then rehydration) of the teeth. With difficult intrinsic stains, the thicker portions of the tooth usually lighten less (i.e., gingival third lightens less than the body and incisal thirds).
7. If the dentist anticipates that swallowing or aspiration of an appliance may occur, treatment should be avoided.
8. When tray is not in service, it should always be stored in appliance case provided with each Opalescence kit.
9. Dark, nonvital, endodontically treated teeth are rapidly whitened by opening lingual access of tooth. The patient is given an Ultradent® syringe with a Black Micro® tip and instructed to irrigate lingual opening with water, then fill opening with Opalescence. The tray is then loaded and inserted. Evaluate periodically, as nonvital teeth can whiten significantly in 1-4 days.

#### Procedure:

1. Fabricate the tray using laboratory instructions or send the working cast/model, and prescription order form (enclosed in kit) to Ultradent.
2. Use patient instructions provided to instruct patient on the bleaching procedure, cleaning, application of Opalescence, and tray care. Explain treatment regimes to patient:
  - a. Most dentists prefer nighttime bleaching regimes for their patients for maximum results with less bleaching material use. Salivary flow decreases during sleep. The lowered mouth activity prevents "pumping" of material from tray. Opalescence 10%, 15%, and 20% carbamide peroxide gel actively bleaches 8-10 hours during the night or 4-6 hours during the day.
  - b. Alternative, customized treatments can be used for as few as 30 minutes to multiple hours per day, depending on patient's needs, level of sensitivity, and day-to-day activities.
  - c. Stopping for multiple nights does not jeopardize continued bleaching when reinitiated.
3. Show patient how to load tray by expressing one continuous bead of gel approximately half way up (or slightly lower) from the incisal edge on the facial side of the tray from molar to molar. This should use no more than 1/3 to 1/2 of a syringe. Instruct patient to clean tray with soft brush and cool tap water after each use.
4. Occasionally tooth sensitivity and/or gingival irritation may require treatment. First treatment of choice is application of UltraEZ®, a potassium nitrate, sticky, viscous gel that may be worn in the Opalescence bleaching tray from 1-2 hours to overnight as needed. Alternative treatments include the following:
  - a. Patient may wear tray with a near-neutral, sticky, viscous, bland fluoride gel (Flor-Opa®). Daytime use insulates teeth from cold, allowing sensitivity to subside.
  - b. Appropriate administration of mild anti-inflammatory analgesics (e.g., ibuprofen or aspirin).
5. It is standard to evaluate patients every 3-5 days of treatment. The clinician may schedule checkups more or less frequently, depending on patients' needs and degree of progress. The number of days required for complete treatment mainly depends on the type of stains present and their severity. For example, tetracycline stains will require more treatment time than other types of disfiguring stains.

*Ultradent Products, Inc.  
Attn: Tray Fabrication  
505 West 10200 South  
South Jordan, Utah 84095*

#### Precautions:

1. Read and understand all instructions. Use recommended tray design and administer appropriate treatment regime to maintain optimal patient comfort.
2. Gingival and general oral health should be good prior to initiating treatment.
3. As a safety precaution, Opalescence should NOT be used during pregnancy or breastfeeding.
4. If patient has a known allergy or chemical sensitivity to peroxides, carbolop, glycerin, etc., Opalescence is NOT to be used. Mint flavor can be irritating to some patients. If patient has a known sensitivity to mint, use alternate flavors.
5. Restorations should be adequately sealed and all exposed sensitive dentin should be covered. If a history of sensitivity exists, treat with appropriate restoration, dentin bonding agent, etc., or temporarily with dentin sealant.
6. Areas of hypocalcification can exist which may not be visible to the naked eye. Hypocalcified areas will whiten faster, thereby becoming

- more obvious during bleaching. Continue bleaching treatment until the unaffected tooth surface blends. Re-evaluate two weeks after bleaching treatment when tooth color has stabilized.
- Monitor patient's progress every few days, depending on severity of tooth discoloration. If only a minor change is desired, evaluate the patient's progress within 1-3 days. Since some teeth become more translucent with bleaching, observe teeth with large metallic fillings closely to prevent show-through.
  - If gingival or tooth discomfort persists (such as excessive temperature sensitivity), inform patient to remove tray, discontinue treatment, and to make an appointment for evaluation of degree of sensitivity.
  - Maintain control of Opalescence whitening agent, dispensing only what is needed to reach the next evaluation appointment (approximately 3 to 5 days). Monitor patients' bleaching progress to prevent bleaching beyond the desired level of whitening or degree of translucency.
  - Understand each patient's expectations before beginning treatment. Inform patients that existing crowns, tooth colored fillings, etc., do not whiten, even though natural teeth potentially will.
  - Opalescence whitening agent is supplied in preloaded 1.2ml unit-dose syringes. Material is NOT to be injected, and patients should be instructed to dispose of syringes when empty.
  - For dentist storage, material should be refrigerated to maintain shelf life. Instruct patients to keep Opalescence out of heat and/or direct sunlight.
  - Patients with heavy occlusion or bruxers may require a thicker tray. We recommend 0.60" tray material.
  - Avoid treatment if patient may swallow or aspirate tray.
  - Instruct patients to be careful not to swallow gel or rinses. Opalescence tooth whitening products contain peroxide and may contain fluoride; swallowing large amounts can be harmful.

#### Laboratory Instructions:

- Pour impression with fast set plaster or dental stone. Alginate must be poured shortly after making impression to ensure accuracy. Trimming is less work if quantity of stone is kept to a minimum. However, have enough mass to ensure removal of model from impression without fracture. Trim base of cast parallel to the occlusal table on model trimmer to within a few millimeters of gingival margin for ease of manipulation. Note palate and tongue areas are removed. Allow model to dry two hours.
- Ultradent® LC Block-Out quickly provides reservoir spaces in tray. Apply ~0.5mm thickness of material onto desired labial surfaces. Stay about 1.5mm from gingival line. DO NOT extend onto incisal edges and occlusal surfaces. This prevents margin of tray from opening upon biting and/or impinging on soft tissues. Patients may experience less tooth discomfort from tray pressures with reservoirs because of reduced orthodontic pressures.
- Cure LC Block-Out for approximately 2 minutes (Ultra-Lume®). A hand-held intraoral light can be used (~20-40 seconds per tooth). Wipe off oxygen inhibition layer.
- With vacuum former (UltraVac™ or use Econo Vacuum Former), heat tray material until it sags approximately 1 inch. Activate vacuum and adapt softened plastic onto model. Cool and remove model.
- Cut excess bulk of material away with serrated plastic trimmers (Ultradent® Utility Cutters).
- With small tactile scissors (Ultra-Trim™ Scalloping Scissors), carefully and precisely trim tray 0.25 to 0.33mm occlusal from gingival margin. Scallop the gingival embrasures to prevent covering the gingival papillae with the tray. The papillae may become sore and irritated if tray covers them!
- Return tray to model; check tray extensions. Gently flame polish edges, one quadrant at a time (Blazer® Micro Torch).
- While still warm, hold periphery of each segment firmly against model for three seconds with water-moistened finger. If an area is short of the desired length, gently heat and push the tray material to the desired location. If this over thins the tray material, a new tray should be fabricated.
- Once tray is fabricated clean with a soft brush and cool tap water. Store tray in appliance case when not in use.

*Stay shy of gingiva, scalloping around interdental papilla.*

*Lapping onto gingiva may irritate tissues.*

Step 1

Step 2

Step 3

Step 4


Step 5

Step 6


Step 7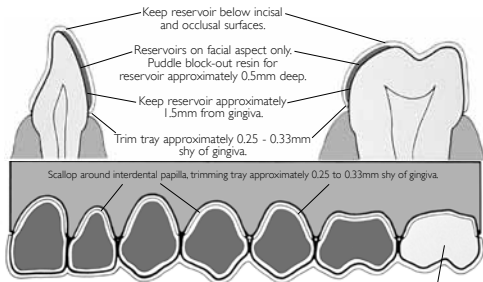

If one or more teeth are much darker than others (e.g., overly dark cusps), build reservoirs on cusps only to balance end whitening results.

Teeth of lower esthetic concern need not receive resin.

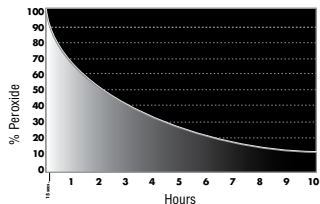*Sustained Release Test: After dental examination and prophylaxis, Opalescence-filled customized tray (plastic membrane) was inserted. Small quantities of gel were removed at various intervals throughout the night for analysis of percentage of active ingredient.*

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#### Bleaching Precautions:

1. Do not use household bleach to whiten your teeth.
2. Do not use Opalscence while pregnant or breastfeeding.
3. Keep gel out of heat/sunlight. Do not freeze.
4. Tooth fillings and crowns do not respond to tooth bleaching.
5. Foods and juices high in acid may cause sensitivity.
6. If you have any questions regarding the appropriate use of this product, please consult your dentist as soon as possible.
7. Do not use tobacco products or eat while bleaching.

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#### General Precautions:

1. For Professional use only.
2. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
3. Keep products out of heat/sunlight.
4. Avoid skin exposure to resins.
5. Isolate strong chemicals to area of treatment.
6. Confirm that patient has no known allergies to treatment materials.
7. Test flow of materials from syringe and tip before using intraorally.
8. Never force syringe plungers.
9. Clean and disinfect syringes between patients.

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**Medicinal Ingredients:** Carbamide Peroxide (10%, 15%, 20% w/w), Potassium Nitrate (0.50% w/w), Sodium Fluoride (0.25% w/w) (1100ppm F)

**Ingédients médicinaux:** peroxyde de carbamide (10%, 15%, 20% g/g), nitrate de potassium (0,50%) fluorure de sodium (0,25% g/g) (1100ppmF)

**Non-Medicinal Ingredients:** Carbomer, Flavour: mint, melon, none, Glycerine, Polyethylene Glycol, Sodium Hydroxide, Water, Xylitol, Edetate Disodium.

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Shelf life of gel: 24 months  
Refrigerate to maintain shelf life.

For professional use only.  
Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line,  
refer to Ultradent's catalog or call Toll Free: 1-800-552-5512.

Outside U.S. call (801) 572-4200  
[www.ultradent.com](http://www.ultradent.com)

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Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers.

The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date.

The first two numbers are the month, and the third number is the last number of the year.

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Imported By:

Clinicians Choice Dental Products /  
Clinical Research Dental  
N6A 1M6

© Copyright 2009 Ultradent Products, Inc. All Rights Reserved.

U.S. Patent Nos.: 5,098,303; 5,234,342; 5,376,006; 5,409,631; 5,667,386; 5,725,843; 5,746,598; 5,759,037; 5,759,038; 5,770,105; 5,770,182;  
5,846,058; 5,851,512; 5,855,870; 5,985,249; 6,036,943; 6,086,855; 6,183,251; 6,306,370; 6,309,625; 6,368,576; and 6,730,316.

Australian Patent No.: 654955. European Patent No.: 0522087. Japan Patent Nos.: 3113182 and 2909040. Canada Patent No.: 2078776.

Other U.S. and International Patents Granted and Pending.

505 West 10200 South  
South Jordan, Utah 84095, USA

**ULTRADENT**  
PRODUCTS, INC.

#5511-CN.4 062410