## EDTA

18% Solution

### Description & Indications:

Ultradent EDTA is an 18% colorless, low-viscosity, aqueous, dentin-chelating solution. It contains a surfactant to facilitate better cleaning. It assists in root canal preparation by decalcifying dentin and removing the smear layer. It is recommended to use EDTA as a final rinse before obturating canal with EndoREZ. EDTA should also be used alternately with sodium hypochlorite between each file during canal instrumentation.

### Preliminary preparation steps:

Prepare IndiSpense® System delivery:

1. An empty 1.2ml or 5ml syringe is attached to the end of the large, no-waste IndiSpense syringe by turning the Luer Lock of the unit dose syringe snugly onto the male thread of the IndiSpense as shown in Fig. 1.
2. With palm grasp (Fig. 2.), the plunger of the IndiSpense syringe is depressed while the unit dose syringe is stabilized at the plunger with the non-dominant hand. Slightly pull out on plunger of unit dose syringe to facilitate loading. Do not express contents of unit dose syringe back into IndiSpense syringe. To prevent cross-contamination, a used syringe should not be reattached to the IndiSpense syringe.
3. Remove Luer Lock cap from syringe. Twist a NaviFX™ Tip or NaviTip™ firmly onto syringe. Gauge NaviFX Tip with the rubber stop at least 2mm short of the root terminus as show in Fig. 3.

### Procedure:

1. Make sure NaviFX Tip is loose in canal and at least 2mm short of terminus. Gently deliver the EDTA with NaviFX Tip. Move brush tip up and down to agitate and brush solution, cleaning walls with EDTA as shown in Fig. 4, alternating with sodium hypochlorite after each file during instrumentation. Do not express EDTA when canula tip is at full length or if it fits tight against the canal walls.


*Figs. 1 & 2 A 1.2ml unit dose syringe is twisted firmly onto the IndiSpense syringe. While supporting plunger of unit dose syringe with the non-dominant hand, depress plunger of IndiSpense and fill unit dose syringe to desired level.*

1. Perform a final rinse with clean, bacteria-free water, local anesthetic, or plain EDTA for not more than one minute for best results. Quick and effective removal of most of the EDTA solution or other irrigants is facilitated by using a Luer Vacuum Adapter with a Capillary Tip or 10mm Micro Capillary.
2. One or two paper points are usually all that are required for final drying of canals, if canals are first vacuummed out as per step 2 above.
3. Fit master cone, then obturate using EndoREZ<sup>™</sup> hydrophilic sealer and gutta percha, or your preferred obturation method.

### Precautions:

1. Always use a rubber dam for endodontic therapy. Prevent ingestion of EDTA solution.
2. Prevent solution from getting in the eyes or leaving it on the soft tissues for an extended period of time.
3. Do not extend tip past root apex or express EDTA past apex.
4. Do not express EDTA from canula tip at the root terminus or if the tip fits tight against canal walls. This may force solutions past the apex.
5. Do not use this product on patients with a known sensitivity to EDTA.
6. The tips and delivery syringes are disposable and intended for single use.

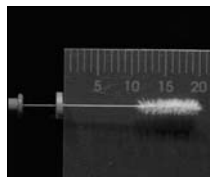*Fig. 3 Measure NaviFX tip 2mm short of root apex and loosely fitted in canal.*

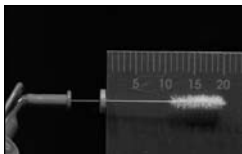
*Fig. 4 Express EDTA into canal. Scrub tip up and down. Rinse canal thoroughly with clean bacteria-free water, local anesthetic or plain EDTA for no longer than one minute.*

Store at room temperature


Measure NaviFX tip 2mm short of root apex and loosely fitted in canal.

Express EDTA into canal. Scrub tip up and down. Rinse canal thoroughly with clean bacteria-free water, local anesthetic or plain EDTA for no longer than one minute.

#### General Precautions:

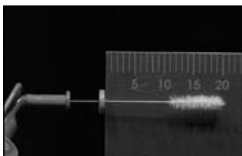1. For professional use only.
2. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
3. Keep products out of heat/sunlight.
4. Avoid skin exposure to resins.
5. Isolate strong chemicals to area of treatment.
6. Confirm that patient has no known allergies to treatment materials.
7. Test flow of materials from syringe and tip before using intraorally.
8. Never force syringe plungers.
9. Use Ultradent® syringe covers and/or clean and disinfect syringes between patients.

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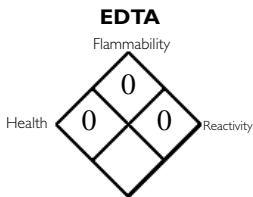
Hazard Rating  
4 = Severe  
3 = Serious  
2 = Moderate  
1 = Slight  
0 = Minimal

**For product MSDS please see our website: [www.ultradent.com](http://www.ultradent.com)**

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

For professional use only.

Caution: Federal law restricts this device to sale by or on the order of a dentist.

Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512.

Outside U.S. call (801) 572-4200  
or visit [www.ultradent.com](http://www.ultradent.com).

© Copyright 2006 Ultradent Products, Inc. All Rights Reserved.