ULTRADENT[®] PRODUCTS, INC.

File-Eze[®]

EDTA Chelating
& Filing Lubricant

Description & Indications:

File-Eze[®] is a 19% EDTA dentin chelating agent in an aqueous, water-soluble solution with a lubricating base.

Indications:

Use File-Eze to aid in instrumentation and debridement of root canals.

Procedure:

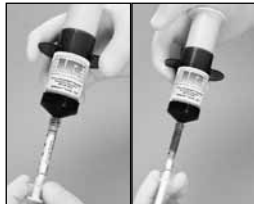
Preliminary Steps:

A. IndiSpense[®] system delivery:

1. The prelabeled unit dose File-Eze syringe is attached to the end of the large, no-waste IndiSpense syringe by turning the Luer Lock of the unit dose syringe snugly onto the male thread of the IndiSpense.
2. Depress plunger of IndiSpense syringe with palm of hand while stabilizing the unit dose syringe at the plunger with the non-dominant hand. Slightly pull on plunger of unit dose syringe to facilitate loading. Do not express contents of unit dose syringe back into IndiSpense syringe. To prevent cross-contamination, a used syringe should not be reattached to the IndiSpense syringe.
3. Place cap on unit dose syringe if not using immediately, or attach a Mini tip and cover with Ultradent's TipSoc[™] covers.
4. When ready for use, continue with procedure steps 1-4.

B. Preloaded 1.2ml syringe delivery:

1. Remove Luer Lock cap from the syringe.
2. Twist a NaviTip[®] firmly onto the syringe. For optimal control, hold syringe with plunger in palm of hand, instead of in conventional manner with thumb on plunger.

Figs. 1 & 2 A 1.2ml unit dose syringe is twisted firmly onto the IndiSpense syringe. While supporting plunger of unit dose syringe with the non-dominant hand, depress plunger of IndiSpense and fill unit dose syringe to desired level.

Procedure:

1. The NaviTip® can be used to deliver File-Eze directly into the canal opening, as shown in Fig. 3. When the canal is difficult to find, start by placing point of file into canal opening prior to introducing File-Eze to pulp chamber (or onto file.) Visibility is not impaired.
2. Once canal opening is engaged, express File-Eze through tip into canal to no more than half the working length. As an alternative, a small amount of File-Eze can be placed directly onto file as seen in Fig. 4.
3. File-Eze is only used with the first two or three instruments.
A thorough amount of sodium hypochlorite is used during entire instrumentation and cleansing procedure, alternating with liquid EDTA between each file.
4. File-Eze is irrigated and rinsed from canal, similar to other root canal preparations; ChlorCid® sodium hypochlorite solution is recommended. A slight effervescent cleansing action occurs when ChlorCid contacts residual File-Eze, which helps remove extraneous material from canal.

Precautions:

1. Deliver only to canal orifice. Do not express material with pressure up canal past apical foramen.
2. Tips are disposed after use.
3. Clean and disinfect syringes between patients.
4. NaviTips do not have a side port hole. Do not express past apex.

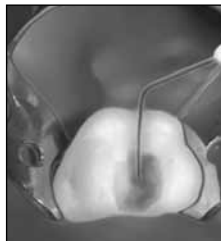
Fig. 3 File-Eze is placed after canal orifice is engaged. Visibility is not impaired.

Fig. 4 File-Eze may be applied directly on file if desired.

ULTRADENT[®]
PRODUCTS, INC.
MATERIAL SAFETY DATA SHEET

Controlled Product Yes No

SECTION I - IDENTIFICATION OF THE SUBSTANCE/PREPARATION AND OF THE COMPANY/UNDERTAKING

Material Name File-Eze [®]	Manufactured by Ultradent Products, Inc. 505 West 10200 South South Jordan, Utah 84095 USA
Material Use Chelating Canal Prep & Lubricant	Emergency Contact 1-800-552-5512 or 1-801-572-4200 (Outside USA) EC Representative
Date January 23, 2006	
Prepared by: Peter Allred	

SECTION II - COMPOSITION/INFORMATION ON INGREDIENTS

Active Ingredients	Approximate Concentration %	LD ₅₀ /LC ₅₀ Specify Species and Route	C.A.S. N.A. or U.N. Numbers
EDTA	19%	LD ₅₀ 330 mg/kg Mouse—Intraperitoneal	10378-23-1 (RTECS refer to C.A.S. Number 64-02-8)

SECTION III - HAZARDS IDENTIFICATION

Adverse Human Effects and Symptoms
Eye irritant.

SECTION IV - FIRST AID MEASURES

Inhalation N/A	Eye Contact Flush with lots of water.
Skin Contact Wash with soap and water.	Ingestion If ingested in large amounts, give large volumes of water.

SECTION V - FIRE-FIGHTING MEASURES

Flammability
Yes No Under Which Conditions:

Means of Extinction N/A	Protective Equipment for Firefighters N/A
----------------------------	--

SECTION VI - ACCIDENTAL RELEASE MEASURES

Personal & Environmental Precautions
Eye protection.

SECTION VII - HANDLING AND STORAGE

Methods for Cleaning Mop up with water.	Handling Precautions General.
	Storage Requirements Store at room temperature.

SECTION VIII - EXPOSURE CONTROLS/PERSONAL PROTECTION

Respiratory Protection N/A	Eye Protection Safety glasses.
Hand Protection N/A	Skin Protection N/A

SECTION IX - PHYSICAL & CHEMICAL PROPERTIES

Appearance Milky white cream or gel	Solubility in Water Soluble	Melting Point/Range N/A
Odor None	pH ~10.3	Other Data N/A

SECTION X - STABILITY AND REACTIVITY

Chemical Stability Conditions and Materials to Avoid Yes <input checked="" type="checkbox"/> No <input type="checkbox"/>	Hazardous Decomposition Products Ammonia may be generated.
---	---

SECTION XI - TOXICOLOGICAL INFORMATION

Route of Entry <input type="checkbox"/> Skin Contact <input type="checkbox"/> Skin Absorption <input type="checkbox"/> Eye Contact <input type="checkbox"/> Inhalation Acute <input type="checkbox"/> Inhalation Chronic <input checked="" type="checkbox"/> Ingestion
Effects and Symptoms of Short-Term Exposure Eye: Irritation.
Chronic Effects and Symptoms of Long-Term Exposure Skin: Irritation. Eye: Irritation. Ingestion: Irritation.
Sensitization N/A
Initiation of Product Slight.

SECTION XII - ECOLOGICAL INFORMATION

Biodegradability
Not defined.

SECTION XIII - DISPOSAL CONSIDERATIONS

Method of Waste Disposal
In accordance with government regulations.

SECTION XIV - TRANSPORT INFORMATION

Not defined.

SECTION XV - REGULATORY INFORMATION


Not defined.

SECTION XVI - OTHER INFORMATION


FOR DENTAL USE ONLY. Use as directed.
 The information and recommendations are taken from sources (raw material MSDS(s) and manufacturer's knowledge) believed to be accurate; however, Ultradent Products, Inc., makes no warranty with respect to the accuracy of the information or the suitability of the recommendation and assumes no liability to any user thereof. Each user should review these recommendations in the specific context of the intended use and determine whether they are appropriate.

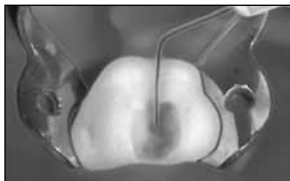Store at room temperature

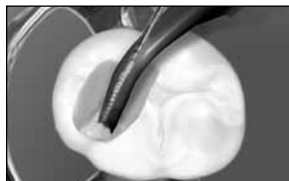
Place File-Eze into canal from bottom up. Following instrumentation, irrigate with sodium hypochlorite.

Alternative: Place directly on file and canal orifice.

General Precautions:

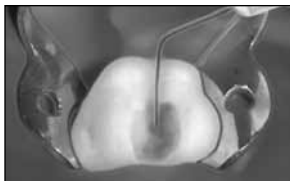1. For professional use only.
2. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
3. Keep products out of heat/sunlight.
4. Avoid skin exposure to resins.
5. Isolate strong chemicals to area of treatment.
6. Confirm that patient has no known allergies to treatment materials.
7. Test flow of materials from syringe and tip before using intraorally.
8. Never force syringe plungers.
9. Use Ultradent® syringe covers and/or clean and disinfect syringes between patients.

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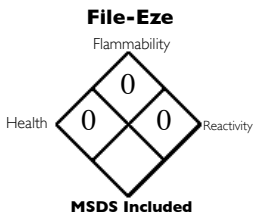
Hazard Rating
4 = Severe
3 = Serious
2 = Moderate
1 = Slight
0 = Minimal

Shelf Life: 36 months. Store at room temperature.

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts the sale of this device by or on the order of a state licensed dental professional.

Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512.

Outside U.S. call (801) 572-4200
or visit www.ultradent.com.

© Copyright 2006 Ultradent Products, Inc. All Rights Reserved.