ULTRADENT'S
OMNI-MATRIXTM
SECTIONAL

Description and Indications:

The new Ultradent Sectional Matrix system is a revolutionary design and advancement in providing anatomically correct contacts for Class II restorations with any restorative material. The unique patented design affords several benefits. There is no need for special forceps, pliers, or numerous size clamps for this system. There are simply four sizes of matrix bands and one size retainer clamp. Standard rubber dam forceps are used for placement. The matrices and clamps are specially designed to provide an anatomical contact of proper size and location. The design and size of the clamp allows them to be stacked/shingled allowing use with multiple surfaces and restorations at the same time.

Procedure:

1. Place a rubber dam and prepare tooth.
Note: make sure interproximal contact is broken for proper matrix insertion.
2. With the rubber dam in place, or with a dental floss tether attached to clamp for control, select proper matrix size.
3. Place contoured matrix in interproximal space and place a properly selected wedge with cotton forceps as shown in Fig.1. The convex edge of matrix should be towards occlusal surface.
4. Place and position clamp or clamps onto tooth with rubber dam forceps using placement slots as shown in fig. 2. Typically the loop of the clamp will be placed toward mesial, however, distal placement is possible if needed. NOTE: Some forcep designs require the handles cross to facilitate adequate opening of the retainer.
5. Ensure proper positioning to cover all cavosurfaces of preparation. Burnish matrix against adjacent tooth to ensure excellent contacts as in Fig. 3.
6. Restore tooth as usual. We recommend using Ultradent's PQ1® followed by PermaFlo® and/or Vit-I-essence™ for maximum function and esthetics. We recommend light curing all surfaces for 20 seconds after matrix and clamp are removed.
7. Remove clamp with rubber dam forceps and matrix band, then remove wedge with cotton forceps or hemostats.

8. Contour, adjust occlusion and polish restoration.

Note: Multiple surface restorations are done in same manner except separate matrices are used for each interproximal surface to be restored with a clamp being placed (stacked) for each matrix as in Fig. 4.

Precautions

1. Placement slots have specifically designed for rubber dam forceps. Placement of clamp on forceps in any other location will result in an inability to open clamp wide enough.
2. Matrices are designed to be disposable, i.e. one use per patient.
3. Clamps should be sterilized following use.
4. We recommend using rubber dam whenever possible. If not, tether clamp with floss to prevent ingestion or aspiration. If a clamp or matrix is ingested or aspirated seek medical attention immediately.
5. Do not over-spread clamp as it will cause permanent deformation.

Fig. 1

Fig. 2

Fig. 3

Fig. 4

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© Copyright 2005 Ultradent Products, Inc. All Rights Reserved.