Opalescence[®]

tooth whitening systems

Opalescence PF 10%, 15%, 20%

Recommended use: Anti-Caries

Usage conseillé: Anticarie

Opal PF 10% NPN 80009556

Opal PF 15% NPN 80008433

Opal PF 20% NPN 80016256

Toothpaste NPN 80002447

EN-Tooth Whitening System

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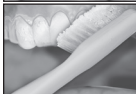
EN-PATIENT INSTRUCTIONS

Load gel into your bleaching tray made by your dentist. Use 1/3 to 1/2 of syringe.


Brush teeth, then insert tray. Adapt tray sides to teeth.
Opalescence 10%, 15%, 20%: Remove tray after 8-10 hours during the night, or 4-6 hours during the day.

Remove excess gel with clean finger or soft toothbrush.
Rinse twice; do not swallow rinses.

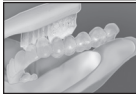Follow dentist's instructions. If significant sensitivity occurs, stop treatment and consult dentist.

Clean tray with soft brush and cool tap water.
Store tray in case provided.

Nonvital Bleaching: Irrigate opening as instructed by dentist. Place gel into lingual access and tray.

Bleaching Precautions:

1. Do not use household bleach to whiten teeth.
2. Do not use Opalescence while pregnant or breastfeeding.
3. Keep gel out of heat/sunlight. Do not freeze.
4. Tooth fillings and crowns do not respond to tooth bleaching.
5. Foods and juices high in acid may cause sensitivity.
6. If you have any questions regarding the appropriate use of this product, please consult your dentist as soon as possible.
7. Do not use tobacco products or eat while bleaching.
8. Do not swallow gel or rinses. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.

General Precautions:

1. Teeth are naturally darker along the gumline. These areas may require more time to lighten than the rest of the tooth surface and usually remain slightly darker.
2. A small percentage of patients experience sensitivity with bleaching. Should this occur, contact your dentist. Your dentist may recommend that you bleach every second or third night. You can also use Opalescence during the day for 1-3 hours.
3. Foods and juices high in citric acid can cause sensitivity to the teeth.
4. Some patients have noticed temporary discomfort of the gums, lips, throat, or tongue. Should any of these symptoms persist more than two days or progressively worsen, call your dentist. These side effects will usually subside within 1-3 days after treatment is discontinued.
5. Coffee, tobacco, and other products can re-stain your teeth over time. Should this occur, the teeth can be re-whitened within a few nights with Opalescence.
6. Regular dental checkups and cleanings are important before and after bleaching to maintain a healthy smile.
7. If you have any questions pertaining to this treatment, please ask your dentist.
8. Some old amalgam or "silver" fillings may leave a dark purple color in your bleaching tray; this is normal.
9. Store bleach out of the sun and heat.
10. Discard any unused bleaching gel after treatment is completed.
11. Keep Opalescence out of reach of small children.

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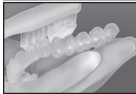
EN - Directions: Adults and children 2 years of age or older: Brush teeth thoroughly, preferably after each meal or at least twice a day, or as directed by a dentist or physician. Instruct children under 6 years of age in good brushing and rinsing habits (to minimize swallowing).

Supervise children as necessary until capable of using without supervision. Children under 2 years of age: Consult a dentist or physician.

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Medicinal Ingredients: Sodium Fluoride (0.25% w/w)

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EN - Warning: As with all fluoride toothpastes, keep out of the reach of children under 6 years of age. If you accidentally swallow more than used for brushing, seek professional assistance or contact a Poison Control Center immediately. Regulations on administering fluoride containing products may differ from country to country.

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Opalescence®

tooth whitening gels

Medicinal Ingredients: Carbamide Peroxide (10%, 15%, 20% w/w), Potassium Nitrate (0.50% w/w), Sodium Fluoride (0.25% w/w) (1100ppm F)

Ingrédients médicinaux: peroxyde de carbamide (10%, 15%, 20% g/g), nitrate de potassium (0,50%) fluorure de sodium 0,11% g/g (1100ppmF).

Non-Medicinal Ingredients: Carbomer, Flavour: mint, melon, none, Glycerine, Polyethylene Glycol, Sodium Hydroxide, Water, Xylitol, Edetate Disodium.

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EN-Toxic
FR-Toxique

505 West 10200 South
South Jordan, UT 84095, USA
www.ultradent.com

All Opalescence products are gluten-free.

Imported By:

Clinicians Choice Dental
Products/ Clinical Research Dental,
N6A 1M6

© Copyright 2009 Ultradent Products, Inc. All Rights Reserved. U.S. Patent Nos.: 5,098,303; 5,234,342; 5,376,006; 5,409,631; 5,667,386; 5,725,843; 5,746,598; 5,759,037; 5,759,038; 5,770,105; 5,770,182; 5,846,058; 5,851,512; 5,855,870; 5,985,249; 6,036,943; 6,086,855; 6,183,251; 6,306,370; 6,309,625; 6,368,576; and 6,730,316. Australian Patent No.: 654955. European Patent No.: 0522087. Japan Patent Nos.: 3113182 and 2909040. Canada Patent No.: 2078776.
Other U.S. and International Patents Granted and Pending. 4431-CN.4 021010