**UltraCal<sup>®</sup>  
XS**

Calcium  
Hydroxide Paste

**Description:**

UltraCal<sup>®</sup> XS is an aqueous, 12.5 pH, syringeable, radiopaque, calcium hydroxide paste. Use it as an endodontic treatment dressing. Superior delivery control of UltraCal XS is obtained with a NaviTip<sup>™</sup> attached.

**Indications:**

UltraCal XS is used for apexification procedures and as an intermediate inter-appointment canal treatment dressing. Calcium hydroxide has been recognized as an antibacterial agent due to its high pH as well as the ability to potentially stimulate healing of bone.

**Procedure:**

Following instrumentation, remove luer lock cap from UltraCal XS syringe and attach a NaviTip. Deliver UltraCal XS short of the apex while slowly withdrawing tip (Fig. 1).

*Note: Use Ultra-Blend<sup>®</sup> plus<sup>™</sup> for pulp capping procedure.*

**Precautions:**

1. Material is basic. Use carefully and rinse with lots of water if material is inadvertently discharged in an undesired area.


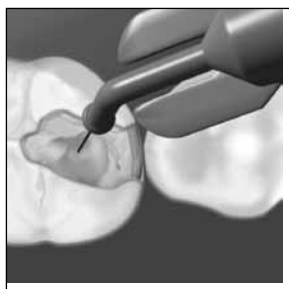
Fig. 1 Following initial instrumentation, canal is filled with UltraCal XS paste.

2. Tips are disposable.
3. Deliver with NaviTip, staying at least 2mm shy of the apex, always keeping the delivery tip loose in the canal. Don't deliver with plastic tips which can occlude tight in the canal (example Capillary<sup>™</sup> tips).
4. Prefilled syringes are multi-dosage. Clean and disinfect between applications as appropriate.
5. We highly recommend using Ultradent<sup>®</sup> Syringe Covers sealed with the Impulse Sealer to provide a protective barrier against cross-contamination.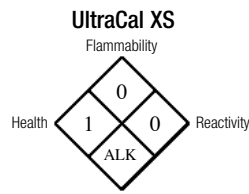

MSDS Included.

**HAZARD RATING**


- 4 = Severe
- 3 = Serious
- 2 = Moderate
- 1 = Slight
- 0 = Minimal

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts this device to sale by or on the order of a dentist.  
Keep out of reach of children. Store at room temperature.  
Shelf Life: 24 months.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200, or visit www.ultradent.com.

<b>UltraCal<sup>®</sup></b>	<b>EN-Calcium Hydroxide Paste</b> <b>DE-Calciumhydroxid-Paste</b> <b>FR-Pâte d'hydroxyde de Calcium</b> <b>NL-Calciumhydroxidepasta</b>	<b>IT-Pasta all'Idrossido di Calcio</b> <b>ES-Pasta de Hidróxido de Calcio</b> <b>PT-Pasta de Hidróxido de Cálcio</b> <b>SV-Kalciumhydroxidpasta</b>	<b>DA-Calciumhydroxid pasta</b> <b>FI-Kalsiumhydroxidi-pasta</b> <b>EL-Πάστα Υδροξειδίου του ασβεστίου</b>	<b>RU- Паста гидроксида кальция</b> <b>PL- Pasta wodorotlenku wapniowego</b> <b>TK- Kalsiyum Hidroksit Pasta</b> <b>AR- لتر اكال أكس أس معجون ماءات الكالسيوم</b>	<b>KR- 수산화칼슘제 임시 근관 충전제</b> <b>CH- 氢氧化钙糊剂</b> <b>TH- สารแคลเซียมไฮดรอกไซด์</b> <b>JA- 水酸化カルシウム配合ペースト</b>
-----------------------------	--	---	--	--	---

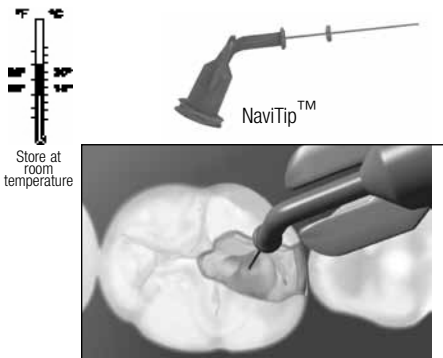EN Following instrumentation, fill canal from apical portion of root outward. Stay at least 2mm shy of apex.

Canal prior to treatment on left, 5 months after application of UltraCal XS on right. UltraCal still in canals.

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## EN General Precautions:

- For Professional use only.
- Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
- Keep products out of heat/sunlight.
- Avoid skin exposure to resins.
- Isolate strong chemicals to area of treatment.
- Confirm that patient has no known allergies to treatment materials.
- Test flow of materials from syringe and tip before using intraorally.
- Never force syringe plungers.
- Use Ultradent<sup>®</sup> Syringe Covers and/or clean and disinfect syringes between patients.
- Use standard curing lights with all light polymerized/activated materials.

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## Ultradent

## MATERIAL SAFETY DATA SHEET

		Controlled Product	Yes <input type="checkbox"/>	No <input checked="" type="checkbox"/>
<b>SECTION I - IDENTIFICATION OF THE SUBSTANCE/PREPARATION AND OF THE COMPANY/UNDERTAKING</b>				
Material Name	UltraCal <sup>®</sup> XS Manufactured by Ultradent Products, Inc. 505 West 10200 South Jordan, Utah 84095 USA			
Material Use	Calcium Hydroxide Paste. Emergency Contact 1-800-552-5512 or 1-801-572-4200 (Outside USA)			
Date	September 29, 2003 EC Representative			
Prepared by:	Peter Allred			
<b>SECTION II - COMPOSITION/INFORMATION ON INGREDIENTS</b>				
	Active Ingredients	Approximate Concentration %	LD <sub>50</sub> /LC <sub>50</sub> Specify Species and Route	C.A.S. N.A. or U.N. Numbers
	Calcium Hydroxide	35% w/w	LD <sub>50</sub> Oral – Rat 7340 mg/kg	1305-62-0
<b>SECTION III - HAZARDS IDENTIFICATION</b>				
Adverse Human Effects and Symptoms:	Basic pH.			
<b>SECTION IV - FIRST AID MEASURES</b>				
Inhalation	N/A	Eye Contact	Flush with lots of water.	
Skin Contact	Wash with soap and water.		Ingestion Drink 2-3 glasses of water.	
<b>SECTION V - FIRE-FIGHTING MEASURES</b>				
Flammability	If Yes, Under Which Conditions:			
Means of Extinction	N/A Protective Equipment for Firefighters N/A			
<b>SECTION VI - ACCIDENTAL RELEASE MEASURES</b>			<b>SECTION VII - HANDLING AND STORAGE</b>	
Personal & Environmental Precautions	None.		Handling Precautions None.	
Methods for Cleaning	Clean with soap and water.		Storage Requirements Store at room temperature.	
<b>SECTION VIII - EXPOSURE CONTROLS/PERSONAL PROTECTION</b>				
Respiratory Protection	N/A	Eye Protection	Protective eyewear.	
Hand Protection	Latex gloves.		Skin Protection Lab coat.	
<b>SECTION IX - PHYSICAL &amp; CHEMICAL PROPERTIES</b>				
Appearance	White paste.		Solubility in Water	Partial. Melting Point/Range N/A
Odor	Odorless.		pH	12.5 Other Data None.
<b>SECTION X - STABILITY AND REACTIVITY</b>				
Chemical Stability	Conditions and Materials to Avoid Strong acids.			Hazardous Decomposition Products N/A
<b>SECTION XI - TOXICOLOGICAL INFORMATION</b>				
Route of Entry	<input type="checkbox"/> Skin Contact	<input type="checkbox"/> Skin Absorption	<input checked="" type="checkbox"/> Eye Contact	<input type="checkbox"/> Inhalation Acute <input type="checkbox"/> Inhalation Chronic <input checked="" type="checkbox"/> Ingestion
Effects and Symptoms of Short-Term Exposure	Eye: Irritation.		Skin: Irritation.	
Chronic Effects and Symptoms of Long-Term Exposure	Eye: Irritation.		Skin: Irritation.	
Sensitization	N/A		Irritancy Slight.	
<b>SECTION XII - ECOLOGICAL INFORMATION</b>			<b>SECTION XIII - DISPOSAL CONSIDERATIONS</b>	
Biodegradability	Yes.		Method of Waste Disposal In accordance with government regulations.	
Further Details	None.			
<b>SECTION XIV - TRANSPORT INFORMATION</b>			<b>SECTION XV - REGULATORY INFORMATION</b>	
Not regulated.			Not defined.	
<b>SECTION XVI - OTHER INFORMATION</b>				
FOR DENTAL USE ONLY. Use as directed. The information and recommendations are taken from sources (raw material MSDS(s) and manufacturer's knowledge) believed to be accurate; however, Ultradent Products, Inc., makes no warranty with respect to the accuracy of the information or the suitability of the recommendation and assumes no liability to any user thereof. Each user should review these recommendations in the specific context of the intended use and determine whether they are appropriate.				