## **ViscoStat® Clear**

### **Description:**

ViscoStat® Clear is a 25% (m/m) Aluminum Chloride gel in a viscous, aqueous vehicle. The pH is ~1. Viscostat Clear is a solution containing aluminum chloride which is used for the temporary hemostasis of the gingival margin during dental procedures such as, but not limited to, dental impressions, seating of temporary and permanent restorations, restorations of cavities and placement of a rubber dam.

### **Indications:**

ViscoStat Clear is a unique hemostatic agent. ViscoStat Clear causes the collagen in the capillary ends to swell and closes off the capillaries. Advantage of this mechanism is to avoid coagulum and hemostatic residue adhering to the preparation. This is especially critical in the “esthetic zone”. ViscoStat® Clear will not stain the hard and/or soft tissues.


## **Tissue Management Procedure for Making Impressions:**

### **A. Hemostasis**

1. An Ultradent 1.2ml unit dose syringe is attached to the end of the ViscoStat, IndiSpense® syringe, (Fig. 1). With a palm grasp

(Fig. 2) the plunger of the IndiSpense syringe is gently depressed while holding plunger on unit dose syringe. Note: To prevent cross-contamination, a new unit dose syringe must be loaded for additional volumes.

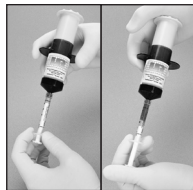2. Twist a Dento-Infusor tip securely onto the syringe. Check evenness of flow before dispensing intra-orally.
3. Circle the preparation, expressing gel while rubbing until adequate hemostasis is obtained. Rub firmly until all bleeding stops.

*Figs. 1&2*

## B. Cleaning

When hemostasis is obtained, clean preparation thoroughly with a firm air/water spray. Note: A spray is also a test to check for quality/completeness of hemostasis. If bleeding occurs with spray, repeat step 3 and give another air/water spray.

## C. Tissue Displacement

1. Select an appropriate size cord.
2. Press the Ultrapak knitted cord in ViscoStat Clear around the preparation to optimize the gingival tissue displacement.
3. Use the Ultrapak Packer to place the cord.
4. Following cord placement, leave cord in place 4-5 minutes
5. Remove packing cord, rinse with firm air/water spray and dry.
6. Again rub gel into sulcus with Dento-Infusor tip and leave for one minute.
7. Rinse with firm air/water spray. Repeat until gel remains clear. Do a final rinse, dry and make impression.

## Tissue Management Procedure for Direct and Indirect Bonded Restorations:

### A. Hemostasis and/or Displacement:

Rub bleeding area with the ViscoStat Clear in syringe with Dento-Infusor as taught above. Ultrapak knitted cord soaked in ViscoStat Clear and packed into sulcus is also useful for hemostasis and sulcular control. Often the cord can stay in place throughout the restorative procedure. Leaving it in place protects the soft tissues during shaping and polishing of the restoration. Always completely remove upon completion of the restoration.

### B. Cleaning

Always give a firm air/water wash after using the Dento-Infusor or packing cord to ensure no residual hemostatic is present to contaminate surface to be restored. If all hemostatic, saliva, blood, and sulcular fluid is not removed and controlled, contamination of the dentin and/or enamel substrate will occur. This will jeopardize bonding and seal, which will result in “microleakage”. With indirect restorations, temporary cements can also contaminate preparation surfaces.

### C. Place Restorations as usual.

## Precautions:

1. Hemostatics, temporary cements, mucins, and blood will prevent quality adhesion and polymerization/set of resins and will lead to microleakage under any restoration. They must be thoroughly washed using a firm air/water spray and or scrubbing with Consepsis Scrub or pumice.
2. These hemostatic solutions are designed for intraoral use. However, check the evenness of flow before dispensing in the mouth.
3. No practical benefit is gained by using this material with ferric sulfate or other aluminum chloride materials.
4. Do not use on patients with known allergy to aluminum chloride or on patients with chemical hypersensitivities.
5. Always thoroughly and firmly wash hemostatics from tissues before applying other chemistries.
6. Use protective glasses when handling/loading Ultradent materials including these hemostatics.

Subgingival preparation and bleeding sulcus. Rub ViscoStat Clear hemostatic firmly against bleeding tissues.

Ultrapak cord is pressed into gel in sulcus around tooth preparation. Leave 4-5 minutes.

Remove cord. Firm air/water spray. Air dry.

Rub gel into the sulcus again. Leave one minute.

Final rinse, dry, and make impression. Repeat above steps, if needed.

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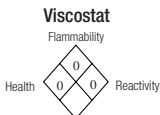
Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

---

For product MSDS please see our  
website: [www.ultradent.com](http://www.ultradent.com)

**HAZARD RATING**

- 4 = Severe
- 3 = Serious
- 2 = Moderate
- 1 = Slight
- 0 = Minimal

---

**Contains no epinephrine. U.S. federal law restricts this device to sale by or on the order of a dentist.**

**For professional use only. Keep out of reach of children. Viscostat Clear shelf life: 18 months.**  
For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free **1-800-552-5512**. Outside U.S. call (801) 572-4200, or visit [www.ultradent.com](http://www.ultradent.com).

© Copyright 2004 Ultradent Products, Inc. All Rights Reserved.  
U.S. Patent Nos.: 5,635,162; & 5,785,955. Australian Patent No.: 726642.  
Other U.S. & Int'l Patents Granted & Pending.

**ULTRADENT**  
PRODUCTS, INC.

**Manufactured by: Ultradent Products, Inc.**  
505 West 10200 South, South Jordan, Utah 84095 USA

#71278.3 032307