

Singles™

UNIT DOSE TIPS (0.3 GRAMS)

ULTRADENT
PRODUCTS, INC.

1. For dentin bonded restorations, first etch using Ultra-Etch® 35%, rinse thoroughly with water and then dry all visible moisture from the tooth surface. Apply PQ1® bonding agent as described in PQ1 instructions.
2. Apply first layer of composite 0.5-1.5mm followed by 2-4mm increments. Use a high-quality curing light to cure each layer 20 seconds.
3. For color matching in posteriors, use shades closer to or slightly lighter than dentin color for deeper layers, then use A1, a Vit-I-escence "Pearl" shade, or other appropriate shades to match enamel for last increment.
4. Final adjustments in shape can be accomplished with multi-fluted carbide burs. Green, yellow, and blue Jiffy® points, disks, and cups can be used for polishing. Follow with PermaSeal® composite sealer.

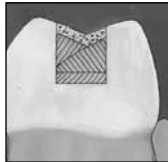


Fig 1. First apply bonding agent such as PQ1. Incremental layering of composite assures complete cure and minimizes post-op sensitivity from polymerization shrinkage.

Note: Vit-I-escence Singles fit most composite guns.

Precautions:

1. Keep tip caps on until use.
2. Make sure tip is firmly snapped into composite gun before trying to express.
3. Unit dose singles are designed for single use only. Discard tips after use.
4. Any resin can be sensitizing to skin or can cause allergic reaction if individual is already sensitized. Therefore, wash area of skin exposed to resin completely to remove material. If dermatitis, rash, or any other allergic reaction is observed, see a physician.
5. Placing posterior composite correctly is much more technique-sensitive than for amalgams. Remember to polymerize interproximal areas adequately and to check radiographically at regular checkup intervals.

Refrigerate if not used on a daily basis. Shelf life 60 months.

For additional information see Vit-I-escence instruction REF/UP #20850, call Ultradent Products, Inc. 1-800-552-5512.